


Urine Monovette® / Urine Monovette® with Stabiliser - User Guide


1

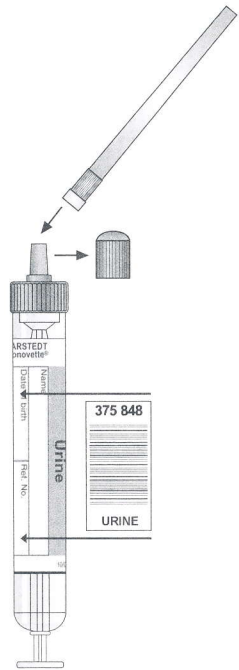

Remove stopper
and keep for later use!


Attach plastic straw.

2

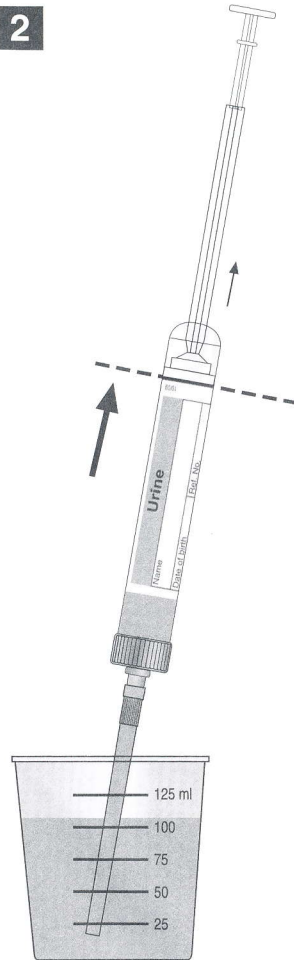


Insert the plastic straw into
the container and fill the
Urine Monovette® up to
the bottom of the label.

3

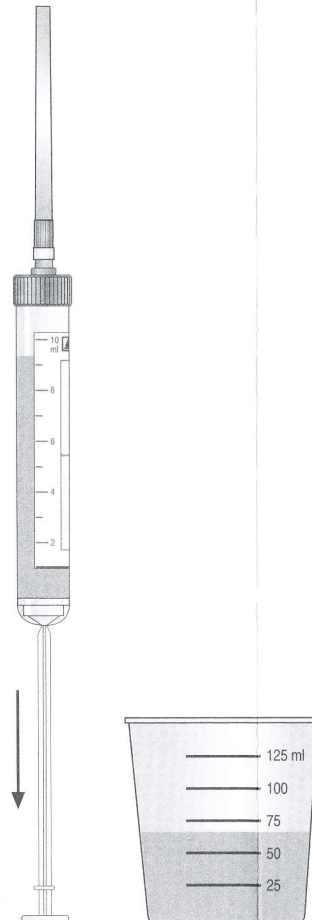

To empty the plastic straw, hold
the Urine Monovette® in an upright
position and pull the plunger
backwards to the bottom of
the tube.


4

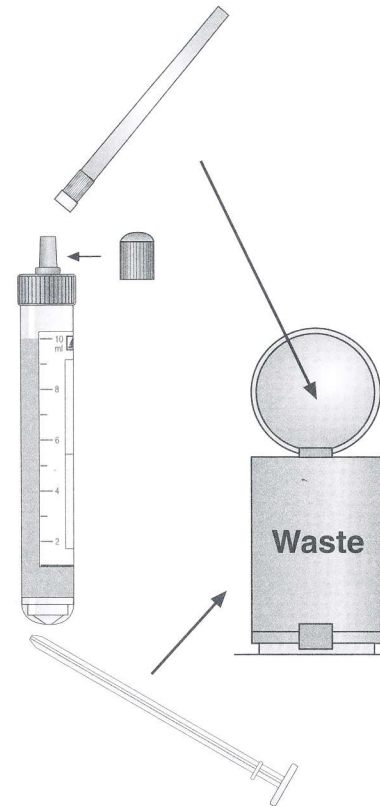

Remove the plastic straw, break
off the plunger and throw away.

Replace the stopper.

5

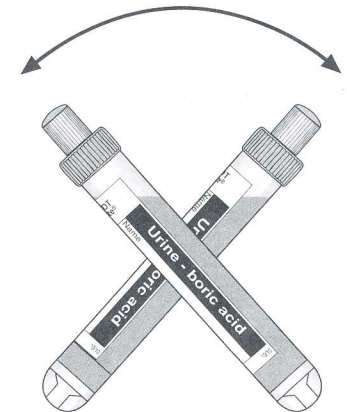

When using the Urine Monovette®
with stabiliser, mix well after
sample collection (tilt approx. 5 times).